

nüm

nüm quick before the stick




1

Swab the treatment area with antiseptic per facility protocol.



2

Twist to break tamper evident label and pull off base cap.

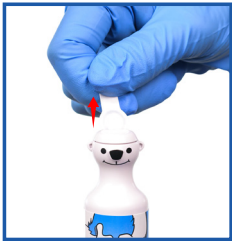

3

Pull off sterile barrier tab on nozzle.



4

From 3-6 inches away spray the treatment area for 4-6 seconds until the skin just turns white.

Do not frost the skin.



5

Immediately perform procedure.

Numbing effect can last for up to 1 minute.

Common uses for nüm™

- Injections
- Vaccinations
- Blood Draws
- IV starts (Venipuncture)
- Port access
- Biopsies
- Minor surgical procedures



*Non-flammable
Non-drug
Rx Only
Sterile
Single-patient use only*

**Patents issued and pending*